Atenschutzmaske

Schutzklasse FFP2 NR

Particle Filtering Half Mask
protection class FFP2 NR

EN149:2001 + A1:2009
Regulation (EU) 2016/425

  **CE 0598**

Art. Nr. DHX9001

Taizhou Donghaixiang Protective Equipment Co. Ltd.
No.3091 Du Nan Avenue Duqiao Town Linhai City Zhejiang
Province China
Tel: +86 (576) 8552 9553

Notifizierte Stelle *Notified body:*
SGS FIMKO OY, Notified Body 0598
Takomotie 8, FI-00380, Helsinki, Finland
t +358 9 696 361, f +358 9 692 5474, www.sgs.com

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FITTING INSTRUCTIONS

1. Hold the particle half mask in position over the nose and mouth.
2. Pull the headharness behind to ears, attach the headharness to the retaining clip, improve comfort and prevent leakage.
3. Ensure the nose clip is securely moulded around the nose, resting the ends against the cheek to obtain a good seal.
4. To check for proper fit, cup both hands over the mask and exhale vigorously. If air leaks around the nose, tighten the nose clip, if air leaks around the edge, reposition the headharness for better fit.
5. Repeat adjustments until the mask is sealed properly.
6. If a proper seal cannot be achieved do not enter the contaminated area or it may cause illness.

USER INSTRUCTIONS:

Failure to follow the instructions and warning on the use of this mask during all times of exposure can reduce the effectiveness of the mask and could result in illness or disability. A copy of the Declaration of Conformity can be found on our website at the following download link: www.ksi-werbeartikel.de/DHX9001. As with any respiratory device, the wearer must be adequately trained prior to use. Before use, the wearer should always check the mask is in good condition - no dirt, no damage to headharness attachment etc.

FIELD OF APPLICATION:

FFP2 masks provide protection against solid and liquid harmful dusts, fumes and aerosols. FFP2 protection class respirators are suitable for work environments where harmful and mutagenic substances are present in the air breathed. They must capture at least 94% of airborne particles up to a size of 0.6 µm and may be used when the occupational exposure limit is no more than 10 times the concentration.

LIMITATION:

Do not use the respirator or enter or stay in a contaminated area under the following circumstances:

1. Atmosphere contains less than 19.5 % oxygen.
2. If you smell or taste contaminant.
3. For protection against gases or vapors.
4. Contaminants or their concentrations are unknown or immediately dangerous to life or health.
5. For sandblasting, paint-spray operations and asbestos treatment.
6. In explosive atmospheres.

WARNINGS:

1. This mask marked "NR" is for single use only.
2. Never substitute, modify, add, or omit parts in the configuration as specified by the manufacture.
3. This mask helps protect against certain particulate contaminants but does not completely eliminate exposure to the risk of contracting disease or infection.
4. It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.
5. Change the mask immediately if breathing becomes difficult or the mask becomes damaged or distorted.
6. Change the mask if a proper face seal cannot be achieved

STORAGE:

1. Store the mask in a sealed container away from contaminated areas when not use.
2. Store the mask in the original packing during transport.

Protection level: FFP2 NR with degree of protection $\geq 94\%$

Shelf life: intended for single short-term use; when stored in closed bag: 2 years.

Art. No.: DHX9001 | Storage: dry, <80% humidity, -20°C to +25°C