

COVID-19 Antigen Rapid Test (Colloidal Gold) anterior nasal - self testing device

- Specimen: Anterior nasal swab
- Result time: 15mins
- Shelf life: 24 months
- Storage temperature: 2-30°C
- Sensitivity: 96.1%
- Specificity: 99.2%



Test Preparation




- A Test Kit
- B Instruction For Use
- C Specimen Collection Swab
- D Test Cassette
- E Extration Buffer Tube
- F Waste Bag

What you should do before starting the test?


- 1 \ Open the test kit, check components;
- 2 \ Read the instrucion;
- 3 \ Check the expiration date;
- 4 \ Disinfect your hands thoroughly ;
- 5 \ Open the package containing the sterile swab.

Specimen Collection

COLLECTION BY AN ADULT CAREGIVER SELF COLLECTION

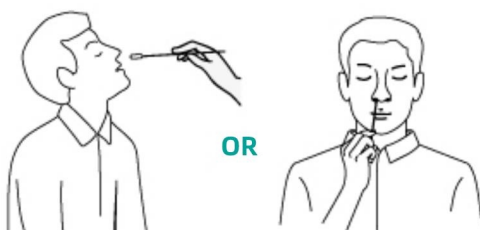


1 ~ 3

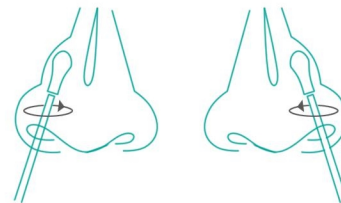

1. Gently insert the entire soft end of the swab about 2 cm into one nostril.
2. Using medium pressure, slowly rub the swab in a circular motion around the inside of your nostril 4 times for a total of 15 seconds.
3. Repeat the same procedure with the same swab in the other nostril.


Test Procedure and Interpretation

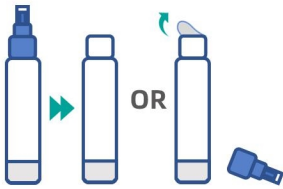


1. Open the cap or aluminium film of the extraction tube.




2. put the anterior nasal swab with the sample into the extraction tube and rotate it 10 times.

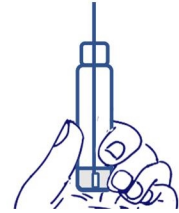


3. Squeeze the wall of the extraction tube by hand.

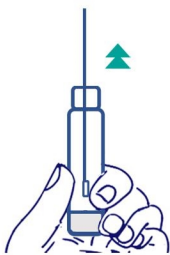

4. Take out the swab.

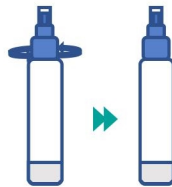


5. Cover the cap of the extraction tube, set aside.

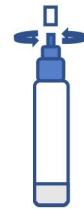


6. Unscrew the little transparent cap.



7. Take the test cassette from the packaging bag, place it on a table.

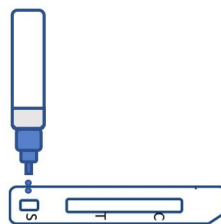

8. Add **2 DROPS** of the sample into the sample hole vertically.

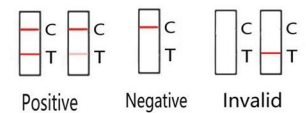

9. Read the result after 15 minutes. If left unread for 20 minutes or more the results are invalid and a repeat test is recommended.

Ordering Information

Product Name	REF No.	Specimen	Package
	FGCOVG7100	Anterior nasal swab	1 Test/kit
COVID-19 Antigen Rapid Test (Colloidal Gold) anterior nasal - self testing device	FGCOVG7200	Anterior nasal swab	5 Tests/kit
	FGCOVG7300	Anterior nasal swab	10 Tests/kit
	FGCOVG7400	Anterior nasal swab	25 Tests/kit

CONTACT:

E-mail: info@joinstar.cn Tel: +86-571-89023160 Fax: +86-571-89028228

Web: <http://en.joinstar.cn>

@JoinstarBioTech

@JoinstarBioTech

www.linkedin.com/company/joinstar